

RAPIDREADER

HAND-HELD READER INSTRUCTIONS FOR USE

1. Power On

- Turn on the reader by pressing the on/off switch on the right side of the smartphone.
- Wait as the reader goes through a sequence of startup screens to arrive at the Home Screen.
- Login with the provided User ID and Password to access the Main Menu Screen.




2. Insert Cassette

- Open the reader back cover and slide the test cassette into the tray with the strip visible.
 - Ensure you slide the cassette into the reader tray all the way to the bottom and the right side of the cassette firmly against the right side of the reader tray. Ensure the sample well is at the bottom.
 - Close reader cover and ensure it is firmly shut.
- (Note: if the cassette is incorrectly inserted the reader will not yield a valid result. For example: halfway or partial insertion of cassette in tray; not closing the test cover fully; sample well oriented toward the top)



3. Run Test

- On the Main Menu Screen select 'Run Test'. Select the test program. Input the sample information as needed.
- Choose 'Run Test' at the bottom of the screen. The reader will begin acquiring the digital image of the test.
- Once analysis of the image is complete the test results will be displayed with the test and control (CTL) values.
- The test result will be automatically saved in 'Test Results'. If desired, the saved results can be viewed later by selecting 'Test Results' from the Main Menu.

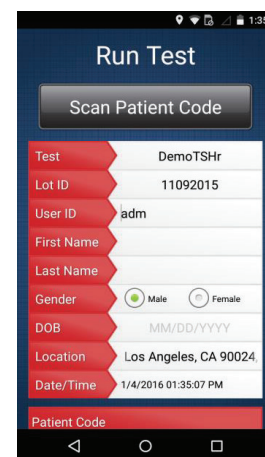

TECHNICAL INFORMATION

Size / Weight	16x8x7 cm / 0.4 kg
Time to Read	<1 min to read
Result Output	Print, email, store on device, transfer via USB cable or send to database via WiFi
Storage Conditions	Do not store in extreme heat, cold or humidity; treat as normal cell phone
Operating Temp	5 to 48 degrees Celsius

Technical support provided by the following authorized distributor...

Biotech Diagnostics PTY LTD

Phone: +61 420 439 299 | **Email:** info@biotechkit.com | **Address:** 2/416 West Street Kearneys Spring, QLD 4350