


1. PREPARATION

- › Always use the pipette included with the kit to ensure the correct volume is used.
- › Use anticoagulated whole blood, serum or plasma in all blood tests.
- › Bring the test and sample to room temperature before running the test, if necessary.

2. SAMPLE

- › Add the sample to the test strip with the pipette vertically without touching the sample window or test strip.

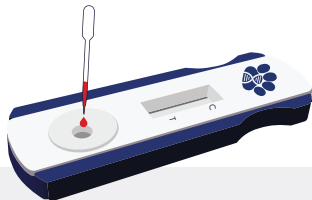


3. ABSORPTION

- › Allow the sample to absorb in the sample window. This could take several minutes.



4. BUFFER

- › Add buffer to the sample well.

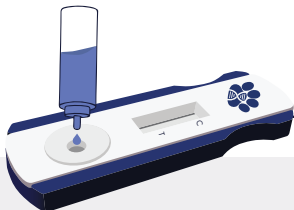

5. WAIT

- › Start the timer and wait 10 minutes.



6. INTERPRET RESULTS

Positive




Both test and control lines are present.

Negative



Only the control line is present.

Invalid

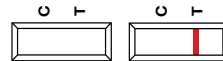

The test is invalid if no line appears at the control line even if a test line is present.

QUESTIONS OR NEED HELP? CONTACT US!

Biotech Laboratories U.S.A. LLC

845-201-3700 | info@btech.vet | www.btech.vet